


EMERGENCY QUALITY WOUND CLOSURE TO GO.

- Be prepared for minor cuts, wherever you may be
- Technology used at hospitals in over 600,000 procedures
- 12X stronger¹ and leaves less scarring² than stitches
- Better wound protection than butterfly bandages and strips

● It's easy to use and effective

The intuitive Zip closure allows you to close minor lacerations in seconds with less pain or puncturing of the skin.

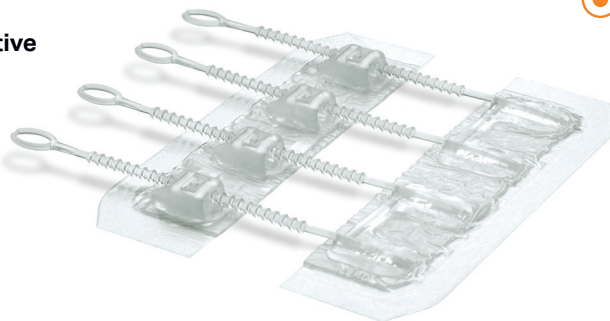

● Save Time

Avoid the hassle and stress associated with emergency rooms and urgent cares. ZipStitch offers you a quick and easy option to treat minor lacerations on the spot.

HOW TO APPLY



FAQ's:

- Specially-designed adhesive lasts for up to 7 days to help ensure your wound heals properly.
- If using a dressing or cover bandage, make sure the dressing adhesive does not contact ANY part of the Zip, including the transparent outer skirt. Otherwise, it may pull the Zip off when the dressing is removed.
- Device is removed by simply peeling off as you would with a bandage.
- You may shower with the ZipStitch device, although it is best to avoid having direct force of the shower contact the device. Do not take a bath, soak in a hot tub, or swim while wearing the ZipStitch device. After a shower, carefully blot the device dry, taking care not to rub the device with a towel.

References:

1. Levi K, et al. Mechanics of Wound Closure: Emerging Tape-Based Wound Closure Technology vs. Traditional Methods. Cureus 8(10); e827. DOI 10.7759/cureus.827.
2. Tanaka, Y. et al. Randomized Study of a New Noninvasive Skin Closure Device for Use After Congenital Heart Operations. Ann Thorac Surg 2016



Oilfield Safety Products
Ph: 780 962-5060

www.lea-der.com

ZipStitch[®]
No needles. No stitches. No hassles.